

# Aespire 7100

Essential performance  
Compact design



Aespire® 7100 shown with Datex-Ohmeda  
CardiCap™/5 monitor and Tec® 7 Vaporizers

## Features

- Enhanced monitor integration capabilities with our Datex-Ohmeda Anesthesia Monitor and Compact Anesthesia monitor
- Lightweight and compact for easy maneuverability
- Optional integrated auxiliary O<sub>2</sub> flowmeter and suction control

## Advanced Breathing System (ABS™)

- One step bag/vent switch turns the ventilator on/off
- Minimal number of parts and tube connections greatly reduces the potential for leaks and misconnects
- Ease of disassembly (no tools)
- Fully autoclavable and latex-free

## 7100 Ventilator

- Volume and Pressure modes with electronic PEEP
- Exhaled volume, airway pressure and inspired oxygen monitoring capabilities
- Direct access to ventilator parameter settings
- Smart alarms direct user to specific problems and affected parameters
- Pressure bar graph for visual reference on a breath-by-breath basis (optional pressure waveform available)

## Improved low flow/reduced life cycle costs

- Only one scheduled maintenance check per year
- Fresh gas flow compensation – automatically (available with tidal volume compensation option)
- Minimum O<sub>2</sub> flow of 50 mL
- Dual air flow tubes standard for higher resolution of low flows



Physical Specifications

Dimensions

Height:	134.5 cm/52.9 in
Width:	72 cm/28.3 in
Depth:	73 cm/28.7 in
Weight:	Approximately 108 kg/238 lb

Top shelf

Weight limit:	34 kg/75 lb
Width:	66 cm/26 in
Depth:	40 cm/15.75 in

Work surface

Height:	81.7 cm/32.2 in
Size:	2160 cm²/334 in²

DIN rail

Side of machine:	34.5 cm/13.6 in
------------------	-----------------

Drawers (internal dimensions)



Height:	17.5 cm/6.9 in
Width:	33 cm/13 in
Depth:	26.5 cm/10.4 in

Absorber bag arm (optional)


Arm length:	30.5 cm/12 in
Bag arm height (adjustable):	87 cm/34.3 in 104 cm/40.9 in

Casters

Diameter:	12.5 cm/5 in
Brakes:	Individual locking front casters

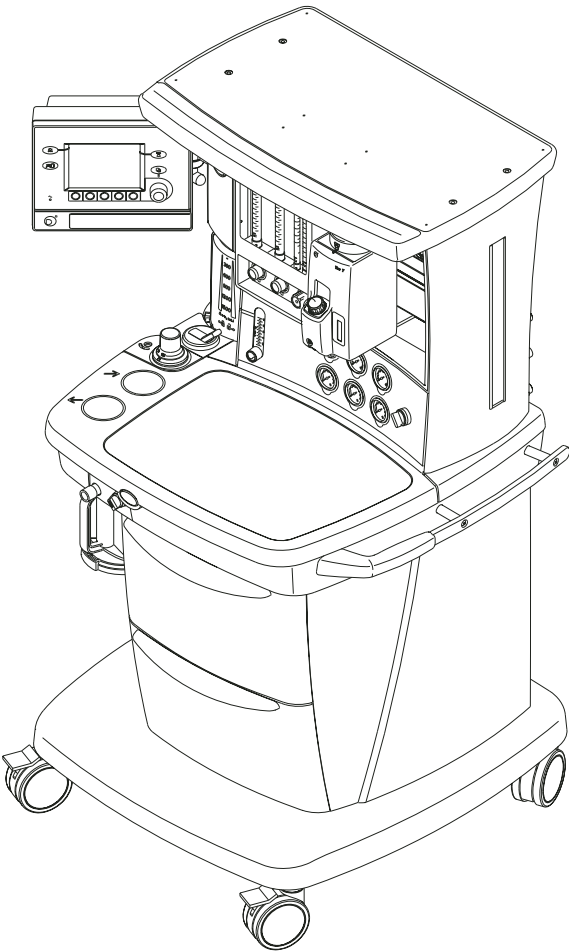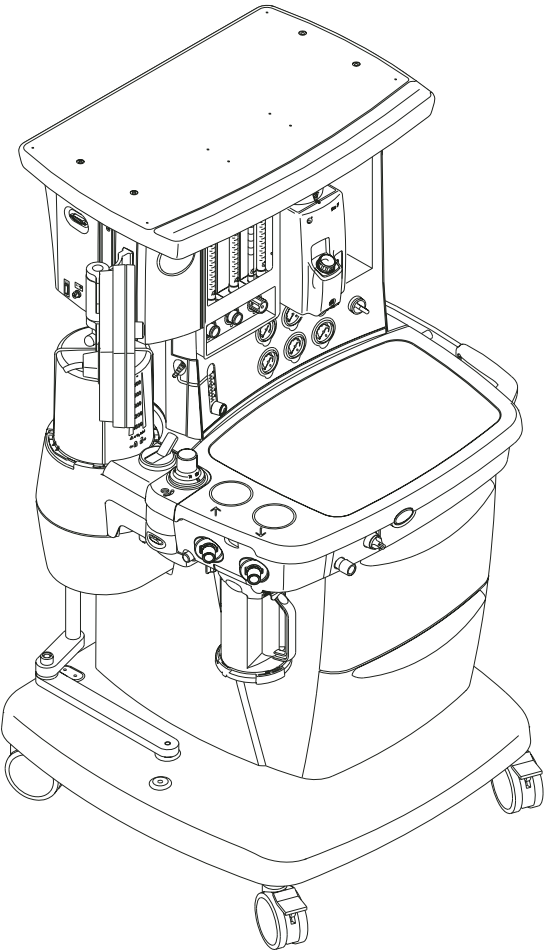

## Ventilator Operating Specifications

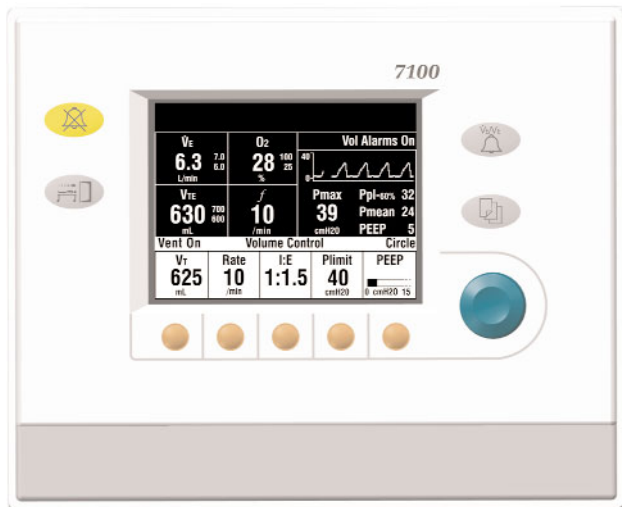

Optional pressure waveform shown

### Modes of ventilation

#### Volume Control mode

With tidal volume compensation (optional)

#### Pressure mode (optional)

### Ventilation parameters

Tidal volume range:	45 to 1500 mL (Volume Control mode)
Incremental settings:	45 to 100 mL (increments of 5 mL)  100 to 300 mL (increments of 10 mL)  300 to 1000 mL (increments of 25 mL)  1000 to 1500 mL (increments of 50 mL)
Pressure ( $P_{\text{Inspired}}$ ) range:	5 to 50 cm H <sub>2</sub> O (increments of 1 cm H <sub>2</sub> O)  5 to 1500 mL volume delivery
Rate:	4 to 65 breaths per minute (increments of 1 breath per minute)
Inspiratory/ expiratory ratio:	2:1 to 1:6 (increments of 0.5)
Inspiratory pause adjust:	5% to 60% of inspiratory time (increments of 5%)

### Positive End Expiratory Pressure (PEEP)

Type:	Integrated, electronically controlled
Range:	OFF, 4 to 30 cm H <sub>2</sub> O (increments of 1 cm H <sub>2</sub> O)

### Ventilator monitored values

Tidal volume:	5 to 1500 mL, 1 mL resolution
Minute volume:	0 to 99.9 L/min, 0.1 L/min resolution
Breathing rate:	0 to 65 breaths per minute, 1 breath per minute resolution
Oxygen percentage:	5% to 110%, 1% resolution
Airway pressure:	-9 to 99 cm H <sub>2</sub> O, 1 cm H <sub>2</sub> O resolution

### Alarm settings

Tidal volume ( $V_{TE}$ ):	Low: OFF, 5 to 1500 mL High: 20 to 1600 mL, OFF
Minute volume ( $V_E$ ):	Low: OFF, 0.1 to 10 L/min High: 0.5 to 30 L/min, OFF
Inspired oxygen ( $FiO_2$ ):	Low: 18 to 100% High: 21 to 100%, OFF
Apnea alarm:	<i>Mechanical ventilation ON:</i> < 5 mL breath measured in 30 seconds  <i>Mechanical ventilation OFF:</i> < 25 mL breath measured in 30 seconds
Low airway pressure:	Change of < 4 cm H <sub>2</sub> O above PEEP
Pressure ( $P_{\text{limit}}$ ) range:	12 to 99 cm H <sub>2</sub> O (increments of 1 cm H <sub>2</sub> O)
Sustained airway pressure:	6 to 30 cm H <sub>2</sub> O + PEEP (adjusted based on ventilator settings)
Subatmospheric pressure:	$P_{aw} < -10$ cm H <sub>2</sub> O
Alarm silence countdown timer:	120 to 0 seconds

## Ventilator Accuracy

### Delivery/monitoring accuracy

Volume delivery:	> 200 mL = better than $\pm 10\%$ Set TV  75 to 200 mL = better than $\pm 20$ mL  < 75 mL = better than $\pm 15$ mL
Pressure ( $P_{\text{Inspired}}$ ) delivery repeatability:	$\pm 2$ cm H <sub>2</sub> O
PEEP delivery repeatability:	$\pm 2$ cm H <sub>2</sub> O
Volume monitoring:	> 200 mL = better than $\pm 10\%$ 75 to 200 mL = better than $\pm 20$ mL  < 75 mL = better than $\pm 15$ mL
Pressure monitoring:	Better than $\pm 2$ cm H <sub>2</sub> O and $\pm 5\%$ of reading (whichever is greater)

## Ventilator Components

### Flow transducer

Type:	Variable orifice flow sensor
Dimensions:	22 mm OD and 15 mm ID
Location:	Inspiratory outlet and expiratory inlet

(Optional autoclavable sensor available)

### Oxygen sensor

Type:	Galvanic fuel cell
-------	--------------------

### Ventilator pneumatics

Pressure range at inlet:	240 kPa to 700 kPa/ 35 psig to 100 psig
Peak gas flow:	70 L/min + fresh gas flow
Flow range:	2 to 70 L/min
Flow compensation range:	200 mL/min to 15 L/min

### Ventilator screen

Display size:	120 mm x 92 mm
Display density:	1/4 VGA standard

### Battery backup

Backup power:	Demonstrated battery time under typical operating conditions is 90 + minutes when fully charged. Battery time under extreme conditions is 30 minutes.
Battery type:	Internal rechargeable sealed lead acid


### Communication port

Serial interface:	Isolated RS-232C compatible port
-------------------	-------------------------------------


## Anesthetic Agent Delivery

### Delivery

Vaporizers:	Tec 5, Tec 6 Plus, Tec 8
Number of positions:	2
Mounting:	Tool-free installation Selectatec® manifold interlocks and isolates vaporizers

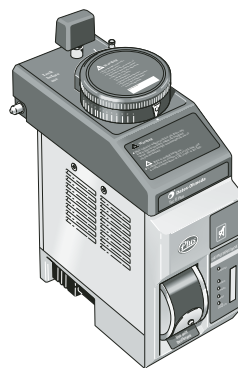

Tec 6 Plus vaporizer

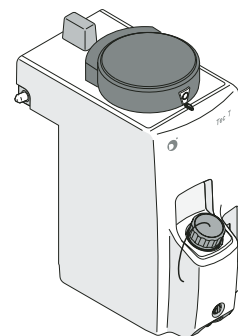

Tec 7 vaporizer

## Electrical Specifications

### Current leakage

100/120 V: < 300 $\mu$ A

### Power

Power input: 100-120 Vac, 50/60 Hz

Power cord: Length: 5 m/16.4 ft  
Rating: 15A @ 120 Vac

### Inlet/outlet modules (120 V)

System circuit breakers: 15A

Outlets (optional): 4 outlets on back, 3-2A,  
1-3A individual breakers,  
optional isolation transformer

## Pneumatic Specifications

### Auxiliary common gas outlet

Connector: ISO 22 mm OD and 15 mm ID

### Gas supply

Pipeline input range: 240 kPa to 600 kPa/  
35 psig to 88 psig

Pipeline connections: DISS-male, DISS-female,  
DIN 13252, AS4059, S90-116  
or NIST (ISO 5359). All fittings  
available for O<sub>2</sub>, N<sub>2</sub>O, and Air,  
and contain pipeline filter and  
check valve.

Cylinder input: Pin indexed in accordance  
with CGA-V-1 or DIN (nut and  
gland); contains input filter and  
check valve

*Note: Maximum 3 cylinders;  
two inboard mounted,  
one outboard mounted.*

Primary regulator  
diaphragm minimum  
burst pressure:

2758 kPa/400 psig

Primary regulator  
nominal output:

< 338 kPa/49 psig  
Pin indexed cylinder connections  
< 407 kPa/59 psig  
DIN cylinder connections

### O<sub>2</sub> controls

Method: Proportionate decrease of N<sub>2</sub>O  
with reduction in O<sub>2</sub> pressure

Supply failure alarm: Range: 193 kPa to 221 kPa/  
28 psig to 32 psig  
Sounds at maximum volume  
every 10 seconds

O<sub>2</sub> flush: Range: 25 to 75 L/min

### Flowmeters

O<sub>2</sub> ranges: 0.05 to 0.95 L/min and  
1 to 15 L/min;

Minimum O<sub>2</sub> flow:  
50 mL/min  $\pm$ 25 mL

N<sub>2</sub>O ranges: 0 to 0.95 L/min and 1 to 10 L/min

Air range: 0 to 0.95 and 1 to 15 L/min

Calibration:	Percent of full scale flow	Accuracy (% of flowrate)
	100	$\pm$ 2.5%
	90	$\pm$ 2.5%
	80	$\pm$ 2.6%
	70	$\pm$ 2.7%
	60	$\pm$ 2.9%
	50	$\pm$ 3.1%
	40	$\pm$ 3.4%
	30	$\pm$ 4.0%
	20	$\pm$ 5.0%
	10	$\pm$ 8.1%

Calibration conditions:\* 20°C/68°F, 101.3 kPa/760 mmHg

\* Different breathing circuit pressures, barometric pressures  
or temperatures change flowtube accuracy.

### Hypoxic guard system

Type: Mechanical Link-25™

Range: Provides a nominal minimum  
25% concentration of oxygen in  
O<sub>2</sub>/N<sub>2</sub>O mixture

### Materials

All materials in contact with patient breathing gases are free  
of natural rubber latex.

## Environmental Specifications

### System operation

Temperature:	10° to 40°C/50° to 104°F
Humidity:	15 to 95% relative humidity (non-condensing) per IEC 68-2-3
Altitude:	-440 to 3565 m/ 500 to 800 mmHg

### System storage

Temperature:	-15° to 50°C/-5° to 122°F
Humidity:	10 to 95% relative humidity (including condensing) per IEC 68-2-3
Altitude:	-440 to 5860 m/ 375 to 800 mmHg
Oxygen cell storage:	-15° to 50°C/5° to 122°F 10 to 95% relative humidity 500 to 800 mmHg

### Electromagnetic compatibility

Immunity:	Complies with all requirements of EN 60601-1-2
Emissions:	CISPR 11 group 1 class B
Approvals:	UL 2601-1, CSA C22.2 #601.1 EN/IEC 60601-1 CE 0197

## Breathing Circuit Specifications

### Operational modes

Breathing circuit is circle mode only

### Carbon dioxide absorbent canister

Absorbent capacity:	800 g
Integrated expiratory limb water reservoir	

### Ports and connectors

Exhalation:	22 mm OD ISO 15 mm ID taper
Inhalation:	22 mm OD ISO 15 mm ID taper
Bag port:	22 mm OD

### Pressure gauge

Scale range:	0 to 10 kPa/-20 to 100 cm H <sub>2</sub> O
--------------	--

### Bag-to-Ventilator switch

Type:	Bi-stable
Control:	Controls ventilator and direction of breathing gas within the circuit

### Integrated Adjustable Pressure Limiting (APL) valve

Range:	0.8 to 70 cm H <sub>2</sub> O
Tactile knob indication at:	30 cm H <sub>2</sub> O and above
Adjustment range of rotation:	0.8 to 30 cm H <sub>2</sub> O (0 to 230°) 30 to 70 cm H <sub>2</sub> O (230 to 330°)

## Materials

---

All materials in contact with exhaled patient gases are autoclavable, except disposable flow sensors and O<sub>2</sub> cell. (Autoclavable flow sensors optional).

All materials in contact with patient gas are free of natural rubber latex.

## Breathing circuit parameters

---

Compliance:	Bag mode:	1.82 mL/cm H <sub>2</sub> O
	Mechanical mode:	Automatically compensates for compression losses within the absorber and bellows assembly
Circuit volume:	2.7 L Vent Mode	
	1.2 L Bag Mode	
Expiratory resistance:	<i>P<sub>exp</sub> Bag Mode</i>	<i>P<sub>exp</sub> Vent Mode</i>
	<i>Flow rate</i>	<i>Pressure drop</i>
	10 L/min	0.78 cm H <sub>2</sub> O
	30 L/min	1.59 cm H <sub>2</sub> O
	60 L/min	3.48 cm H <sub>2</sub> O

Note: With patient circuit and wye piece add +0.89 cm H<sub>2</sub>O

## Anesthetic gas scavenging

---

Type	Hospital system required	Machine connection
Active low flow:	High vacuum 36 L/min (300 mmHg) @ 12 in Hg	DISS evac
Passive:	Passive or externally attached active system	30 mm/0.5 in M ISO taper
Active adjustable flow:	> 30L/min	

© 2005 General Electric Company – All rights reserved.  
GE and GE Monogram are trademarks of  
General Electric Company.

Aespire, Cardiocap, ABS, Tec, Mechanical Link-25 and  
Selectatec are trademarks of Datex-Ohmeda, Inc.

Datex-Ohmeda, Inc., a General Electric company,  
going to market as GE Healthcare.

For more than 100 years, healthcare providers  
worldwide have relied on GE Healthcare for medical  
technology, services, and productivity solutions. So  
no matter what challenges your healthcare system  
faces, you can always count on GE to help you  
deliver the highest quality healthcare. For details,  
please contact your GE representative today.

GE Healthcare  
P.O. Box 7550  
Madison, WI 53707-7550  
USA

[www.gehealthcare.com](http://www.gehealthcare.com)



imagination at work